

## Instructions for Use – please read attentively

Medical  
Device  
under your  
Brand

### 1% medEctoin® Mouth & Throat Spray Salvia

Read this information carefully before you start using 1% medEctoin® Mouth & Throat Spray Salvia. **DO NOT THROW THIS LEAFLET AWAY!** It contains important information.

#### **Intended Purpose:**

1% medEctoin® Mouth & Throat Spray Salvia is a mouth and throat spray for treatment of dry and irritated epithelia in mouth and throat.

#### **Indication:**

1% medEctoin® Mouth & Throat Spray Salvia is indicated for treatment of dry and irritated epithelia in mouth and throat and symptom reduction of inflammation of the mucous membranes in mouth and throat in case of e.g., acute pharyngitis and/or laryngitis, sore throat, common cold, or allergy.

#### **Contraindication:**

1% medEctoin® Mouth & Throat Spray Salvia must not be used in the case of hypersensitivity to medEctoin® or any of the other ingredients of the device.

## 1 What is 1% medEctoin® Mouth & Throat Spray Salvia and what is it used for

1% medEctoin® Mouth & Throat Spray Salvia is a mouth and throat spray for treatment of dry and irritated epithelia in mouth and throat. 1% medEctoin® Mouth & Throat Spray Salvia is a medical device which contains medEctoin®, a natural cell-protective molecule with inflammation-reducing and membrane-stabilizing properties. It reduces symptoms

of acute pharyngitis and/or laryngitis, such as sore throat, hoarseness, cough, and pain on swallowing. Furthermore, 1% medEctoin® Mouth & Throat Spray Salvia hydrates dry and irritated epithelia and supports regeneration by stabilization of the mucosal barrier structure. It supports the reduction of inflammations of the mucous membranes in mouth and throat and reduces the duration of a sore throat or hoarseness. It is recommended to start usage at the first signs of hoarseness, dry cough, swollen lymph nodes and a sore throat. 1% medEctoin® Mouth & Throat Spray Salvia relieves first symptoms such as hoarseness, dry cough, swollen lymph nodes and a sore throat and prevents worsening. Additionally, preclinical data show that the medEctoin® Hydro Complex and the stabilized mucosal barrier structure help to reduce bacterial, viral and allergen ingressions. 1% medEctoin® Mouth & Throat Spray Salvia is a mouth and throat spray with the pleasant taste of Salvia. The spray is free of preservatives.

## 2 Warnings and Precautions

Children younger than 2 years should not use 1% medEctoin® Mouth & Throat Spray Salvia. If allergic reactions occur, the use of 1% medEctoin® Mouth & Throat Spray Salvia must be discontinued immediately. To ensure safe hygiene, each spray bottle should be used by one person only. Do not use for more than 3 months after opening. 1% medEctoin® Mouth & Throat Spray Salvia should be kept out of reach of children. Do not use if the device is damaged or punctured. If you have used a larger amount of 1% medEctoin® Mouth & Throat Spray Salvia than you should, no adverse effects are to be expected if you have used 1-2 single doses too much once. If you have used significantly more and if you experience discomfort, please notify a doctor. The dosage recommendation should be followed.

## 3 What you need to know before using 1% medEctoin® Mouth & Throat Spray Salvia

There are no data available regarding interactions between 1% medEctoin® Mouth & Throat Spray Salvia and other mouth and throat sprays or lozenges, therefore the usage with other mouth and throat treatments is not recommended.

## 4 How to use 1% medEctoïn® Mouth & Throat Spray Salvia

Always use 1% medEctoïn® Mouth & Throat Spray Salvia according to these instructions for use. If not otherwise prescribed by your healthcare provider, please apply 1-2 sprays into the mouth and throat several times a day. A maximum of 10 applications per day should not be exceeded. Before using 1% medEctoïn® Mouth & Throat Spray Salvia for the first time, shake the bottle and pump 2-3 times to release the air from the pump. If the spray function is impaired (which might happen if the bottle is not positioned upright), press the pump mechanism several times in upright position.

1. Shake the bottle.
2. Hold the bottle upright in your hand and place your index finger on top of the spray head.
3. Open your mouth.
4. Hold the spray in front of your mouth and press the spray head down quickly and firmly to spray. Please do not breathe during application of the product.
5. Leave the spray head outside your mouth.
6. If you touched the inside of your mouth or your tongue with the device, wipe the exposed tip of the bottle with a clean lint-free cloth.

## 5 Possible side effects of 1% medEctoïn® Mouth & Throat Spray Salvia

1% medEctoïn® Mouth & Throat Spray Salvia is well tolerated. Up to now, no regularly occurring or persisting side effects are known. Please inform your healthcare provider, pharmacist, or the manufacturer of the product if you notice side effects that are associated with the use of 1% medEctoïn® Mouth & Throat Spray Salvia. If fever occurs, conditions persist for more than 3 weeks or worsen, contact your healthcare provider or pharmacist.

## 6 How to store 1% medEctoïn® Mouth & Throat Spray Salvia

The expiry date of 1% medEctoïn® Mouth & Throat Spray Salvia is printed on the box and on the bottle. 1% medEctoïn® Mouth & Throat Spray Salvia must not be used after the expiry date has passed. 1% medEctoïn® Mouth & Thro-

at Spray Salvia should be stored between 2 °C – 25 °C. The product can be disposed of with household waste.

## 7 Additional information

If recommended by your doctor or health care professional, 1% medEctoïn® Mouth & Throat Spray Salvia may be used after surgery and trauma. Safety of Salvia during pregnancy and while breast-feeding has not been established. In the absence of sufficient data, the use of 1% medEctoïn® Mouth & Throat Spray Salvia during pregnancy and while breastfeeding is not recommended. Children under 10 years and patients unable to use 1% medEctoïn® Mouth & Throat Spray Salvia on their own should be assisted during application of 1% medEctoïn® Mouth & Throat Spray Salvia. Should any serious incident related to 1% medEctoïn® Mouth & Throat Spray Salvia occur, please report it to the legal manufacturer of the product at [vigilance@bitop.de](mailto:vigilance@bitop.de) and the responsible local competent authority.

## 8 Content

20 ml solution

One spray contains 0.15 mL solution

**Ingredients:** 1% medEctoïn®, Sea Salt, Phosphate Buffer, Xylitol, Sodium Cyclamate, Caprylic Acid, Mint Oil, Salvia Oil, Water.

## 9 Marketing Authorisation Holder/Manufacturer/ Distributor

 bitop AG  
Stockumer Str. 28,  
58453 Witten, Germany  
[www.bitop.de](http://www.bitop.de)

 0197

**Date of information:** January 2026

### Symbols



CE mark with Notified Body number



Period after opening



Please do not use if the device is damaged or punctured



Keep out of reach of children (blue)



For re-use on a single patient



Please consider the instructions for use (blue)



Temperature limitation (non-pouched ampoules)



Use by date (Expiry date) or EXP



Manufacturer



Medical Device



Unique Device Identification



Batch Code